# Kashefyar

## Drugs and Psychotropic Substances Identification Rapid Kit

**Application:** Detection of narcotic drugs and psychotropic substances

**How to Use:** The presence of narcotic drugs and psychotropic substances can be screened by color changes of the kit. Based on the type of drugs, the variety of colors produced within 30 seconds.

**Maintenance:** The shelf life of the kit is listed on the package. Keep the kits away from direct sunlight and keep them in the temperature range of  $-20^{\circ}\text{C}$  ( $-68^{\circ}\text{F}$ ) to  $+55^{\circ}\text{C}$  ( $+122^{\circ}\text{F}$ ).

**Accuracy:** This method has a low detection limit of about  $<3\text{ mg}$

### Warnings:


- 1-Please read the instruction carefully before use
- 2-Keep the kit out of reach of children
- 3-Avoid direct contact with skin
- 4- In case of contact with eyes, skin or clothing, rinse immediately with plenty of cold water
- 5-Once your test is complete, dispose all of the used kit contents

**Contents:** 10 Test Ampoules-Ampoule Breakers/10 Spatulas/Instruction Sheet


### How to use

Take the top of the ampoule between the thumb and index finger and snap off the top with the ampoule breaker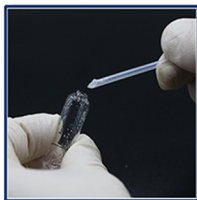

Use the spatula to insert the suspect material into the ampoule

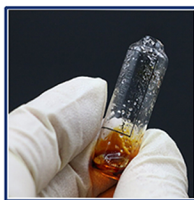Compare the color from the reagent with the color line printed on the pack.

Throw the used ampoules in the trash.

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